

BIO  TECH



BIO TECH
 REST
SLEEP AID DEVICE



Difficult to fall asleep or **wake up easily?**

It is the major form of human sleep distress



Nervousness



Timing of sleep



Daytime working distress



Lifestyle factors such as long-term use of sleeping pills

How do you fix your sleep disorders ?

BIO  TECH



REST



Bio Tech VRest is a microcurrent technology handheld sleep device that mainly helps you fall asleep or improve sleep quality. It delivers low-intensity electrical pulses to the brain via the hands to promote hand-brain synchronisation, which has a positive effect on endorphins, melatonin and other neurotransmitters, helpful for managing anxiety, promote relaxation and help the user fall asleep.



BIO TECH

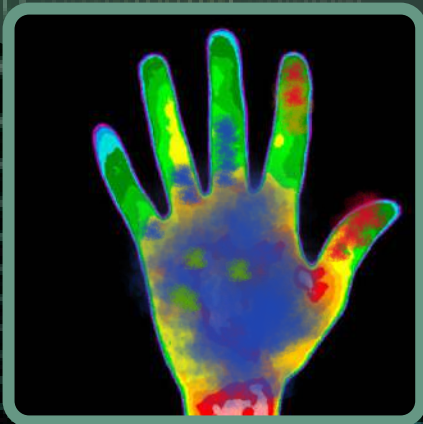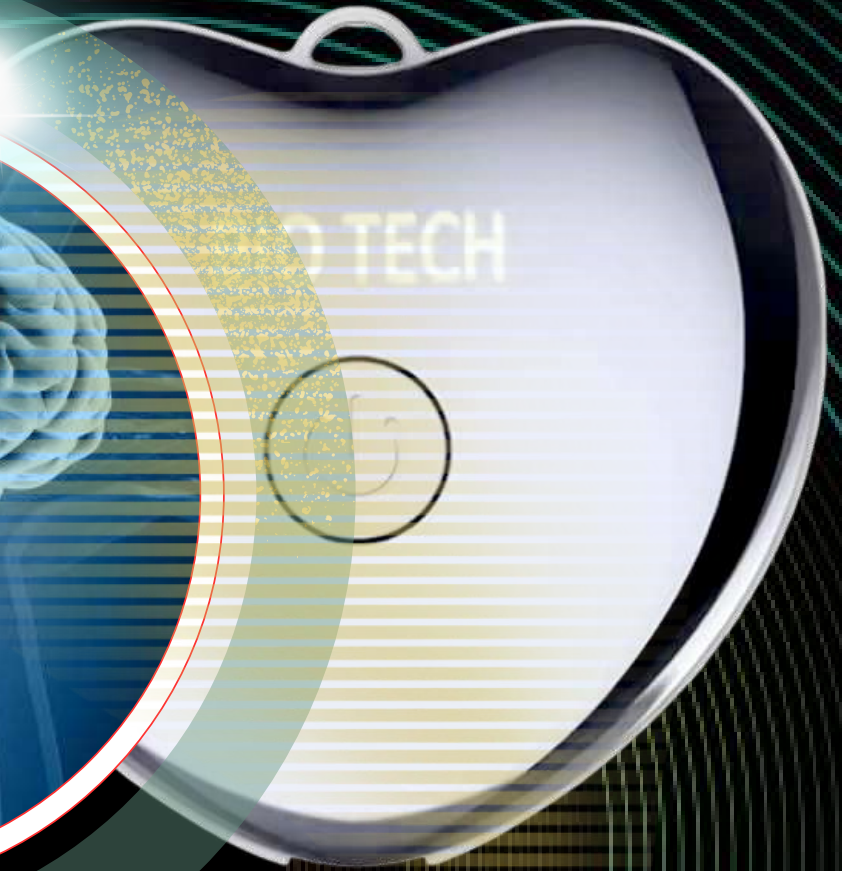


Before




After

The saying that "The Hand Is The Second Brain", this is because a large area of the brain is dedicated to the hand for both sensory and motor control, and the skills we develop with our hands are inseparable from our mental development.


BIO  TECH

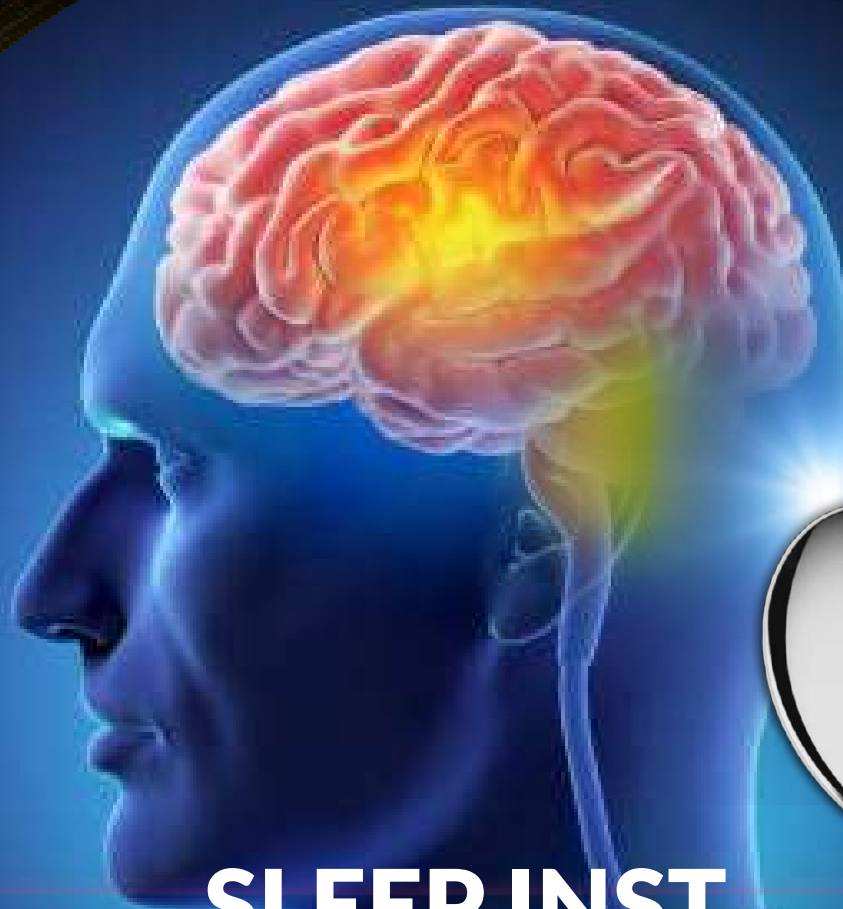

SLEEP INST

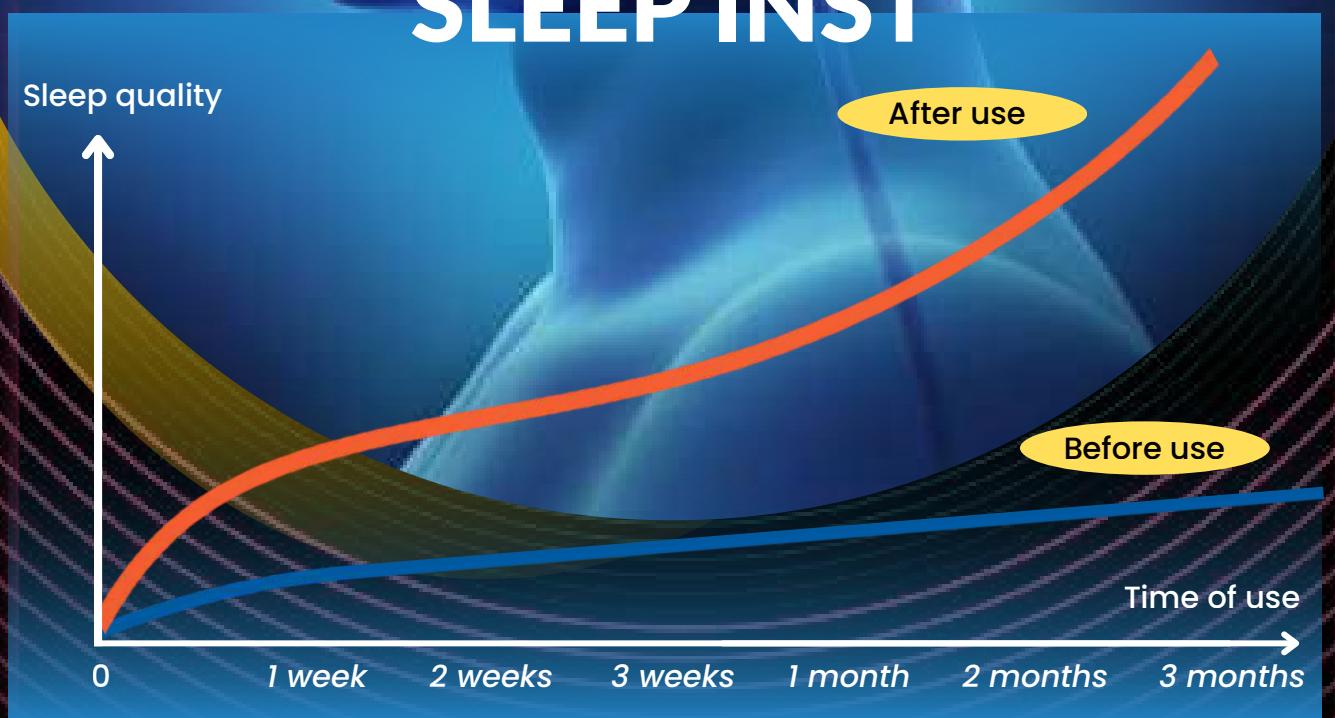

BIO TECH

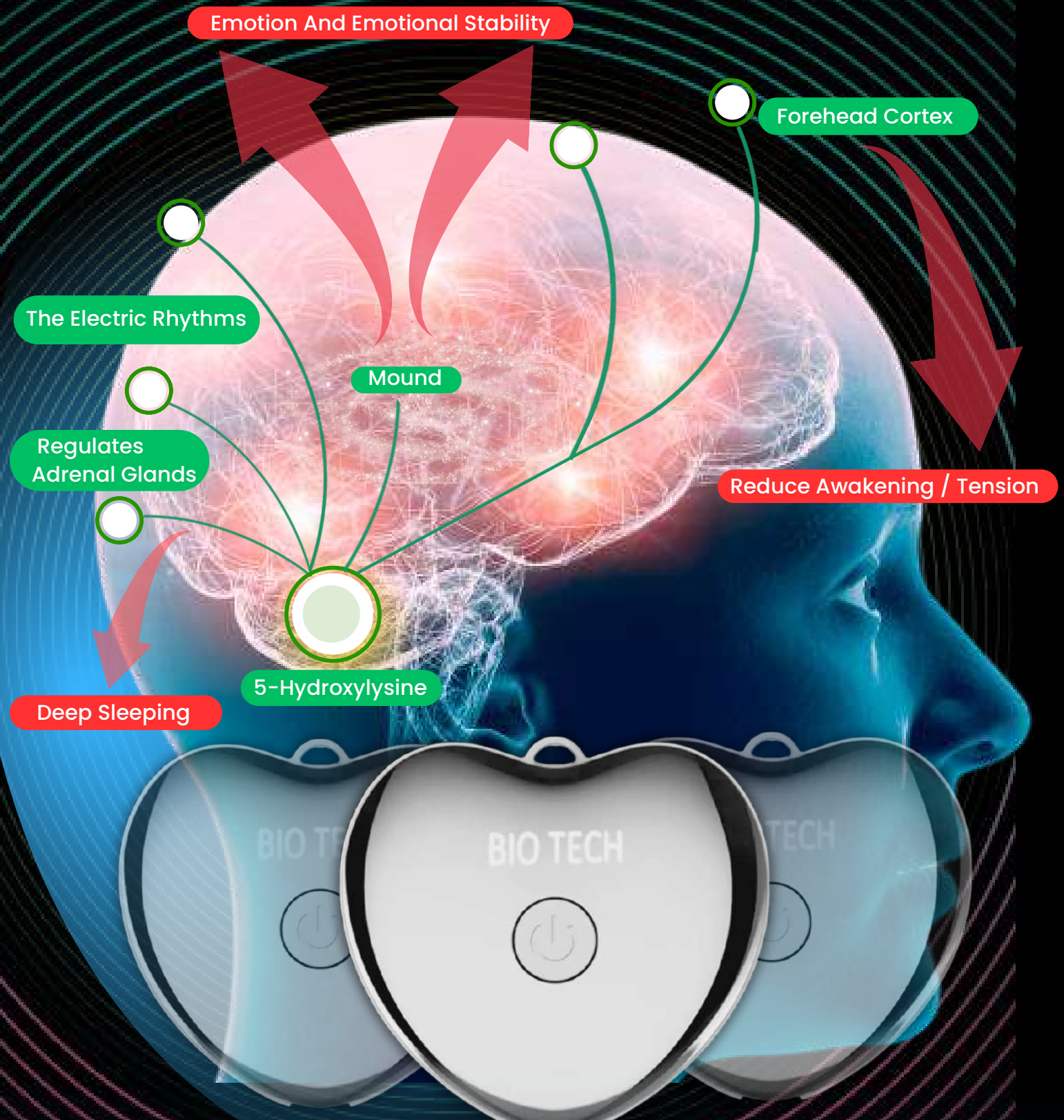

CES Sleep Principle

The CES signal is an 8-15Hz ultra-low-frequency pulse, which is the same frequency as the human brain, it stimulates the secretion of melatonin via pulses through the hand, thereby helping improve sleep and emotion regulation.

PRODUCT FEATURES




PURE PHYSICS

Cranial Electrotherapy Stimulation
(CES)



NON-DEPENDENCE

Non-Medicine, No Side Effects

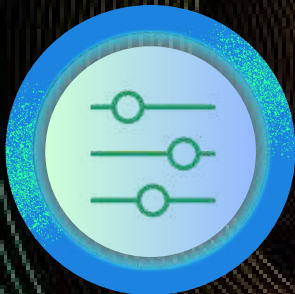

THREE GEARS INTENSITY

Appropriate Power For Sleep And
Excitement Mode



RELAXATION & PORTABILITY

Smart Handheld And Lightweight



INTELLIGENT TIMING PROTECTION

Auto-Shut Down 20 minutes



DECORATIVE DESIGN

Unique And Portable

BIO TECH



Take Medicine

Highly effective

Side effects

Habit forming

Excessive dependence



Insomnia Institutions

Slow effective

Drug dose control

High cost

Uneven improvement



VRest Sleep Aid Device

Fast effectiveness

No Side effects

Low cost

Zero risk



COMPARISON AND WISE CHOICE




AUTHORITATIVE QUALITY CERTIFICATIONS





Product Name: Bio Tech VREST Sleep Aid Device
Product Material: ABS
Product color: Silver or Gun Color
Product Size(mm): 4.32 X 39.3 X 11.4
Packaging Size(mm): 81 X 81 X 17
Packing Contain: Sleep Aid. Lanyard. User Manual. Button Cell
Net Weight: 11.2g
Cross Weight: 45g
Power Supply Method: Button Cell
Battery Capacity: 210mAh
Work Current: 0.5-2.4mA
Working Power: 0.01W
Use Time: The Lowest Gear Test 100 Hours

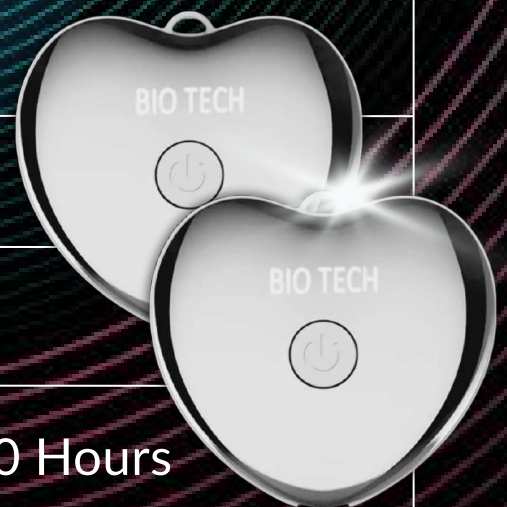

BIO  TECH

Three-Gear Intensity

Select the appropriate gear base on the current needs for power and efficiency



First Gear
blue light



Second Gear
purple red light



Third Gear
red lights

INSTRUCTIONS

Step 1 Prepare the device

- ➡ Press the power button to turn the device on.
- ➡ Press the same button again to cycle through gear.
- ➡ A total of 3 gears, gear circulation to adjust the intensity for appropriate level
 - Low level - blue light
 - Middle level - purple red light
 - High level - red light
- ➡ Long press the power button to shut down the device
- ➡ It has an automatic shutoff feature within 20 minutes
- ➡ When the red light is blinking, you need to replace the button battery




Step 2 Hold and operate the device

- ➡ Hold the device in your hand, with the pulse stimulates therapy to promote hand-brain synchronisation, typically 15-20 minutes
- ➡ For sleep & relaxation, a low-level setting is recommended and shall be held in the LEFT hand



- ➡ For excitement, a high-level setting is recommended and shall be held in the RIGHT hand

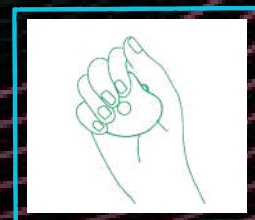

- ➡ Suggest using the device consistently for more than a week to help the body adapt

IMPORTANT CONSIDERATIONS

- ➡ Start with a low intensity and gradually increase it as needed
- ➡ Do not use the device for excessive periods to avoid potential discomfort
- ➡ Keep the device out of the children under 16
- ➡ Consult the doctor if you have a medical condition as below:

ACUTE DISEASES



INFECTIOUS DISEASES



HIGH FEVER



PREGNANT WOMEN



MALIGNANT TUMORS



HEART DISEASE



ABNORMAL BLOOD PRESSURE



BIO TECH



1 *There is no response or feeling after I turning on the device. Why ?*

It may not seem to work because it was on a low setting. You can gradually turn it up to mid or high level until a sensation can be felt in your hand.

2 *Why the device has a sense of electric current ?*

The main function of the device is delivering electrical pulses to stimulate the body's natural relaxation processes. Hence it is normal when you feel the current in your hand.

3 *Why does the device become heated after operation ?*

It is because it contains a built-in heated device which is a functioning part of its operation, which improves comfort further and prevents condensation.

4 *How does the device work ?*

The device delivers mild electrical currents to the brain, which promotes relaxation. Said electrical currents are called alpha brain waves, which can significantly boost your sleep quality by reducing your time taken to fall asleep and increasing the duration of deep sleep.

5 *Does the device suitable for children ?*

The device is intended for users that are above the age of 16.

6 *The device contains another feature called excitement mode, how does it work ?*

It operates at a high-level setting which stimulates neurotransmitters in the cerebral cortex, for which it can help regulate emotions and create feelings of energy and excitement. The left side of the brain is responsible for the body's logical thinking, daily activities and writing skills. It is recommended to be held in the right hand which is controlled by the left brain, therefore boosting qualities of it.

7 *What type of chip does the device contain ?*

Yes. It contains chips as below :

A: The timing and control chips automatically shut off after a default time, adjusting the mode and intensity of the electric current.

B: The microcurrent simulation chip controls the delivery of microcurrent pulses to stimulate the brain, which can help calm the nervous system and promote sleep quality.

C: The heating chip heats and warms up the device, which further improves comfort and prevents condensation.

8 *Is the device safe of use ?*

Microcurrent therapy has been demonstrated to be efficient and safe under real world conditions for the treatment. VRest Sleep Aid Device is generally safe for most healthy users above 16. We recommend users to always start at a low setting, before gradually increasing it to a comfortable level. Use the device for no more 20 minutes per session, and avoid placing electrodes on or near

the neck or eyes. Please consult the doctor if you have a medical condition as below :

- ACUTE DISEASES
- MALIGNANT TUMORS
- INFECTIOUS DISEASES
- PREGNANT WOMEN
- HEART DISEASE
- HIGH FEVER
- ABNORMAL BLOOD PRESSURE

9 *How to replace the button battery when the red light is blinking ?*

Use a screwdriver or other tools to insert the hole of the device bottom, to pry open the bottom shell.



BIO  TECH

REST

LET YOURSELF ENTER THE
DREAMLAND
QUIETLY



www.biotech2328.com